

# **Test Definition: FMELA**

Melatonin, Plasma

## Overview

## **Method Name**

High Performance Liquid Chromatography/Tandem Mass Spectrometry (LC-MS/MS)

#### **NY State Available**

Yes

# **Specimen**

# **Specimen Type**

Plasma EDTA

# **Specimen Required**

Specimen Type: Plasma

Container/Tube:

Preferred: (Lavender top) EDTA

Acceptable: (pink top) EDTA

Specimen volume: 3 mL

Collection instructions: Draw blood in EDTA (lavender top) tube(s). Spin down and send 1 mL of plasma

refrigerated in a plastic vial.

# **Specimen Minimum Volume**

0.5 mL

# **Reject Due To**

-	
Hemolysis	Mild reject; Gross reject
Lipemia	Mild reject; Gross reject
Icterus	NA
Other	Polymer gel separation tube (SST or PST)

# **Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Plasma EDTA	Refrigerated (preferred)	30 days	
	Frozen	365 days	
	Ambient	14 days	



# **Test Definition: FMELA**

Melatonin, Plasma

# Clinical and Interpretive

#### **Reference Values**

Reporting limit determined each analysis.

Units: ng/mL

## Interpretation

Endogenous concentrations of melatonin are of the order of less than 0.02-0.2 ng/mL and vary based on time of day and age. An oral use of a 6 mg dose in 60 female subjects produced an average peak concentration of 12 ng/mL with a peak time of approximately 0.75 hours. A 10 mg dose in male subjects produced an average concentration of 9.8 ng/mL. Melatonin's major side effect profile includes drowsiness and sleepiness.

#### **Performance**

## **PDF Report**

No

# Day(s) and Time(s) Test Performed

Monday-Sunday

# **Analytic Time**

7 days

# **Maximum Laboratory Time**

9 - 11 days

#### **Performing Laboratory Location**

**NMS Labs** 

#### **Fees and Codes**

# **Fees**

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their Regional Manager. For assistance, contact Customer Service.

#### **Test Classification**

This test was developed and its performance characteristics determined by NMS Labs. It has not been cleared or approved by the U.S. Food and Drug Administration.

#### **CPT Code Information**

80375

#### **LOINC®** Information

Test ID	Test Order Name	Order LOINC Value
FMELA	Melatonin, Plasma	11055-1



# **Test Definition: FMELA**

Melatonin, Plasma

Result ID	Test Result Name	Result LOINC Value
Z3521	Melatonin	11055-1